


Grid Pad 10s

Manual

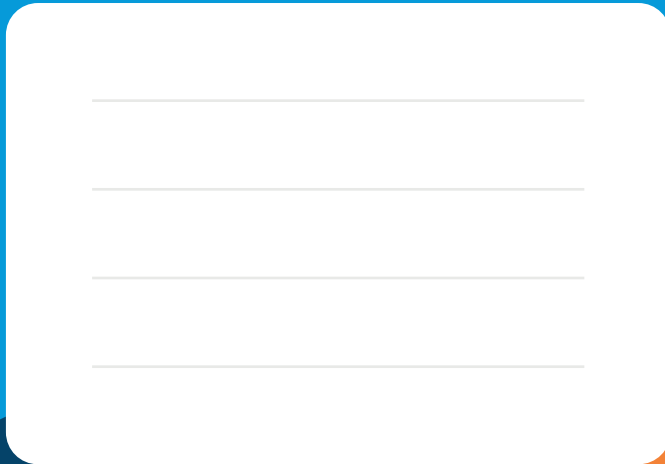

This guide provides the basics for getting started with your Grid Pad.

You can find a complete guide to all the features of your Grid Pad on the Smartbox Hub:



hub.thinksmartbox.com/gridpad10s

Smartbox

Document: Grid Pad 10s - Manual - UK - D
Language: English
Date of issue: October 2022

Copyright © 2022 Smartbox Assistive Technology

Grid 10s

Manual

1.	Starting Grid 3	4
2.	Features	6
3.	Ports	8
4.	Power	9
5.	Charging	11
6.	Remote Power Button	12
7.	Mounting	13
8.	Strap	14
9.	Finding your model and serial number	15

1

Starting Grid 3

Grid 3 will automatically start when the device is switched on by default, however you can start Grid 3 from the desktop from the Grid 3 icon on the desktop.



For more resources and guides on how to get started with Grid visit the Smartbox Hub.



- Get started with Grid
- Access methods and setup
- Troubleshooting guides
- Videos and webinars
- Resources for grid sets
- ...and much more.





hub.thinksmartbox.com/grid3

2

Features

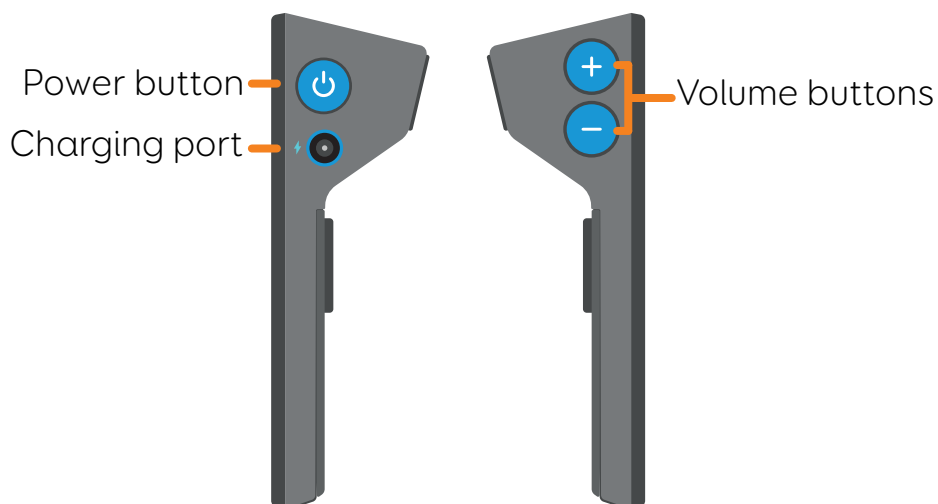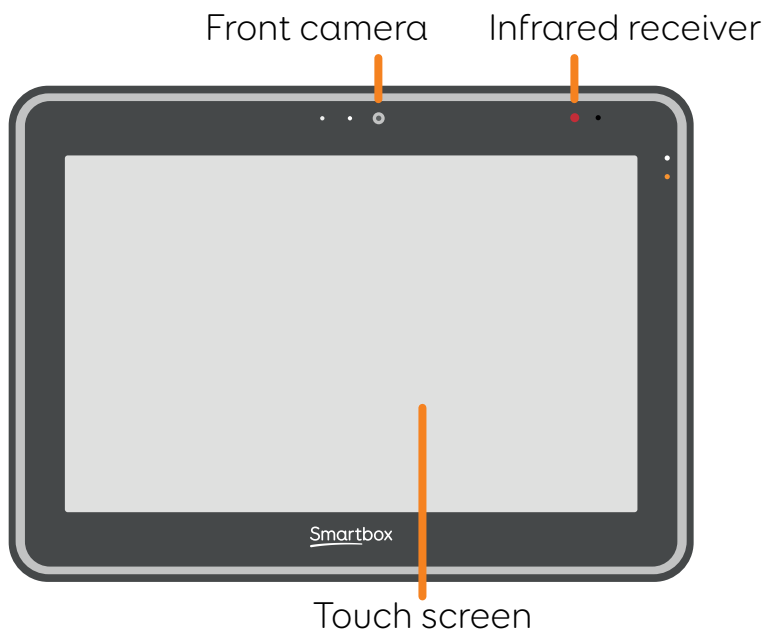


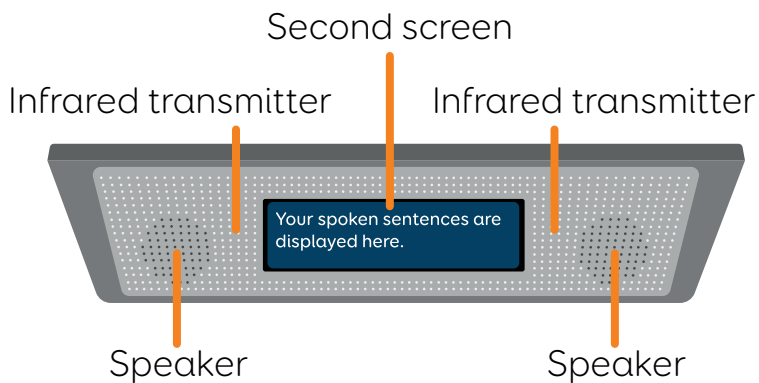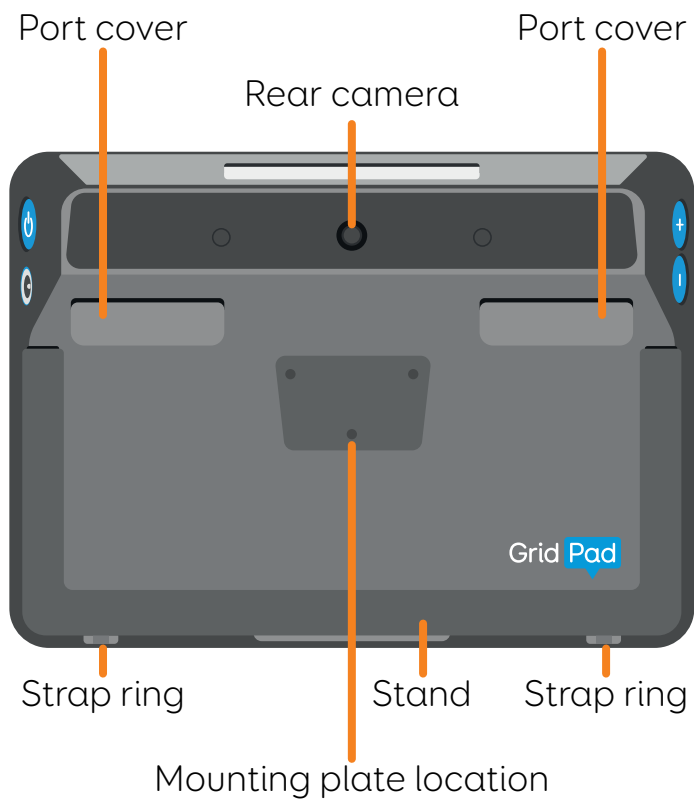

3

Ports


Your Grid Pad has a number of ports to connect extra devices and access methods to. These are located under the **port covers**.

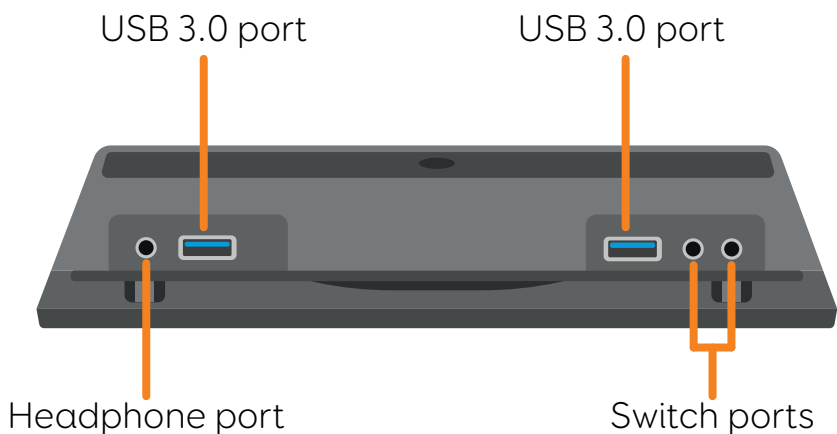

Power

4

To **power on** your Grid Pad **press the power button** on the side of the device for just under a second.

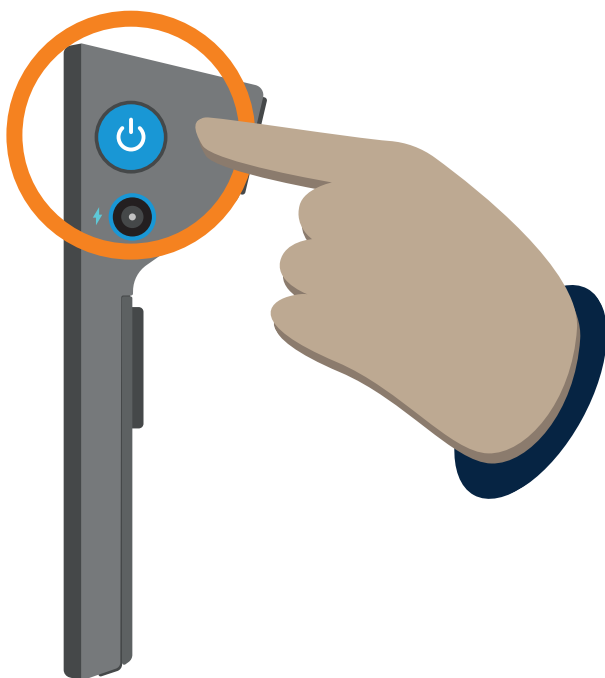

A white light on the front of the device will come on and the system will start up.

If the device is in **sleep mode** it will **wake up**.

You can power off your Grid Pad in several ways.

Windows shut down

Close any apps you have open and make sure you have saved any work.

Tap the **Start menu**

Select the **Power** icon

Select **Shut down**

The Grid Pad will shut down and any pending Windows updates will be installed.


You can also **Restart** your device from here, which will apply any updates.

Charging

5

You will need to charge your Grid Pad's battery after use.

Connect your **Grid Pad charging port** to the **power adapter**, the adapter is clearly labelled.

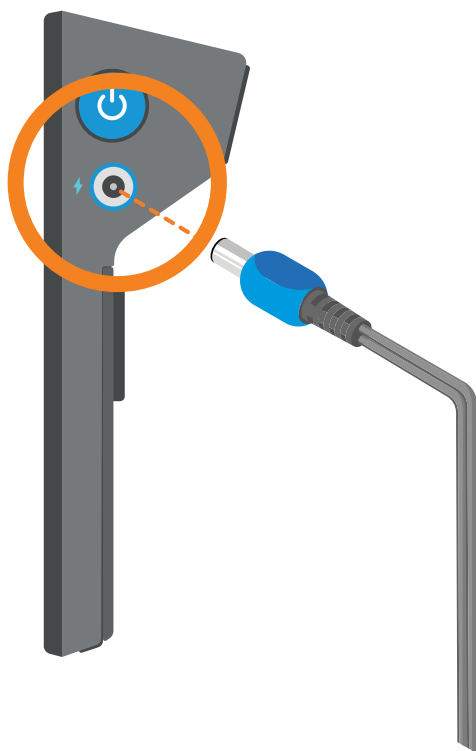

Connect the **power adapter** to the **mains socket** to charge.

6


Remote Power Button

Your Grid Pad has a **Remote Power Button** included.

The Remote Power Button allows you to **turn your Grid Pad on and off**.

It can also be used to **setup switches away from the device**.

The Remote Power Button comes paired with your Grid Pad out of the box.

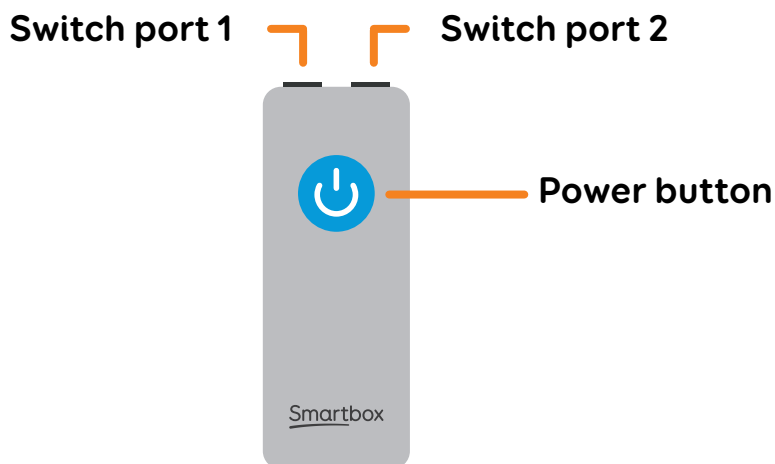


Switch port 1 also functions as a power button.

Mounting

7

The Grid Pad mounting kit is compatible with **Daessy** and **ReHadapt** mounts.


Use a screwdriver to firmly attach the mounting plate to your Grid Pad. Ensure all three screws are secure.

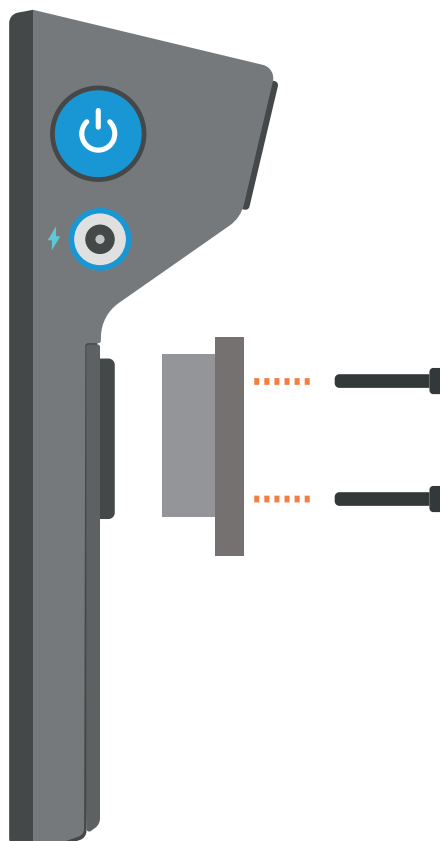

8

Strap

You can attach the Grid Pad strap to the built in strap rings on the bottom of the device.

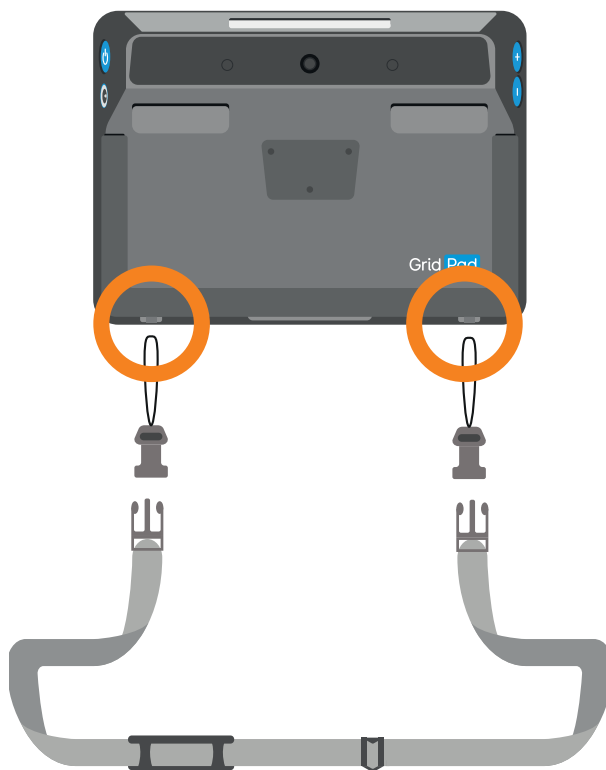

Thread the clip strings through the rings, then pull the plastic clip through the string. Attach the strap to the clips.


Finding your model and serial number

9

When you contact our Support team, please have your device model and serial number to hand. This helps us identify the device you are having an issue with!

You can find the serial number on the back of your Grid Pad.

The serial number for the device is placed under the stand, lift up the stand and the serial number is found here:

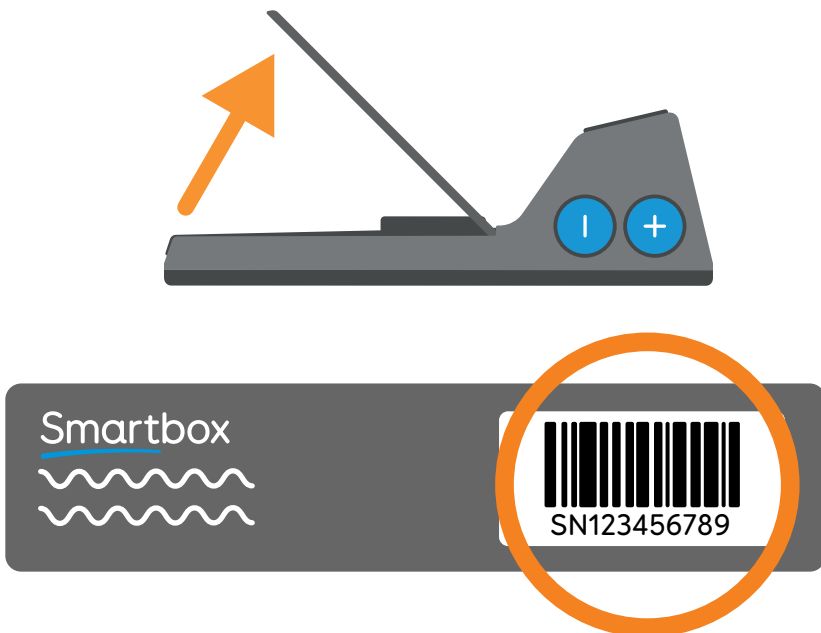

Grid Pad 10s



hub.thinksmartbox.com/gridpad10s

Smartbox

Grid Pad 10s - Manual - UK - D