

L 10

Standard Edition

Manual

Smartbox

Communication is life

This device is manufactured by Smartbox Assistive Technology Inc.

United States

Smartbox Assistive Technology Inc
167 Hillcrest Shopping Center
Lower Burrell
PA, 15068

Document: L 10 (10.2) - ST - Manual - US - A
Device: TP10E-L10
Language: English
Date of issue: September 2023

Copyright © 2023 Smartbox Assistive Technology

L - Series

L 10 Standard Edition

This guide provides the basics for getting started with your L 10.

You can find a complete guide to all the features of your L 10 on the Smartbox Hub:




hub.thinksmartbox.com/l-series

1. Setting up your device


Step 1

Power on your L 10 with the power button on the left side of the device.

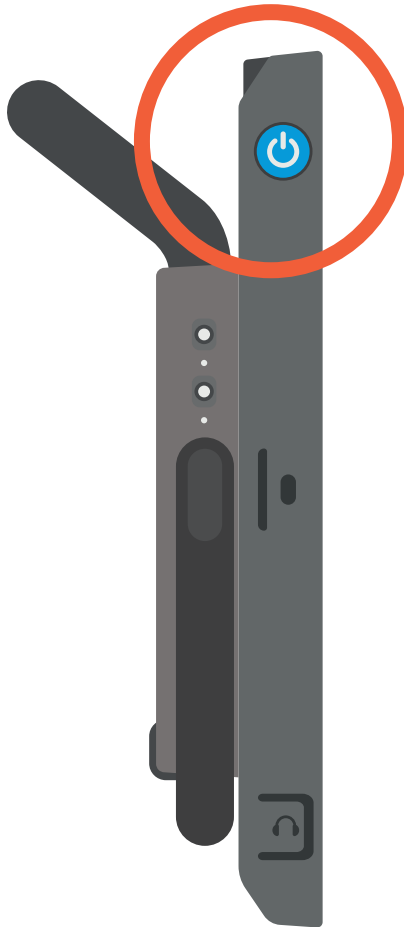

Step 2

Power on the speaker by pressing and holding the Connect button for 3 seconds.

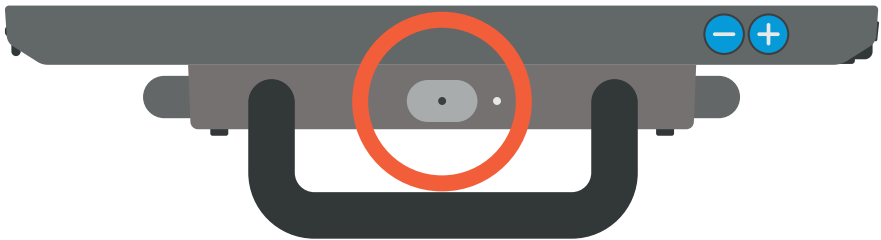

Step 3


L 10 is usually provided Ready to Go.

A short guide is provided with details about your configuration.


Remember to take note of any account details!

2. Power

The device power button is found on the top left side of your L10.

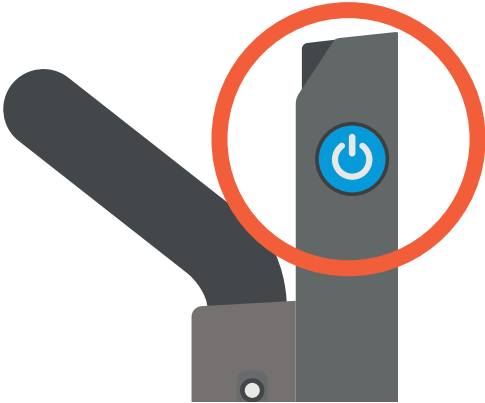

To turn on your device on, press and hold the power button for 3 seconds and press and hold the Connect button for 3 seconds.

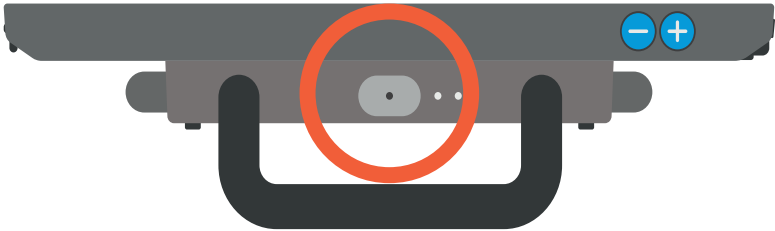

Putting your L 10 to sleep

Sleep is a low powered mode where the device is inactive, and the screen is off.

Tap the power button to sleep the device.

Tap the power button again to wake the device.

Power off

To turn the device off, press and hold the power button.

Swipe the 'Slide to power off' toggle on the screen. Press the cross to cancel.

Press and hold the Connect Button for 3 seconds to power off the speaker.

3. Charging


When charging your L 10, ensure that the device and the speaker are charged at the same time. The L 10 charger has a Lightning cable and a USB C cable.

Connect the L 10 power adapter to a mains socket.

Connect the charging cable to the power adapter.

Connect the Lightning cable to Charging port 1.

Connect the USB C cable to Charging port 2.

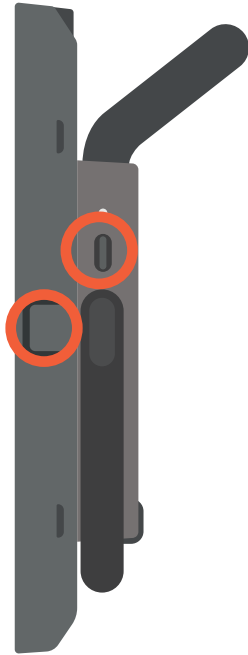

It is best to charge your L 10 when not in use.

4. Bluetooth and pairing

Depending on your L 10 setup, the speaker will either be paired when you unpack your device, or you will need to pair the speaker as part of the setup process.

To pair the speaker, make sure the speaker is off (press the Connect button for 3 seconds).

Press and hold the Connect button for 10 seconds to power on and start pairing mode.

The Bluetooth settings are found in the Settings app. Select Settings, then Bluetooth to see a list of all connected devices. The speaker is listed as **Smartbox AAC - XXXX**.

Press and hold the Connect Button for 3 seconds to power the speaker off.

Tip: The code following Smartbox AAC matches the number printed on the right side of the speaker, under the stand.

5. Using the stand


The stand for your L 10 can be used to position it on a desk or table so it can be used comfortably at any angle.



6. Mounting


Your L 10 can be mounted to a Rehadapt or Daessy connection (Mounting plate sold separately).

The dual mounting plate can be attached with the screws and screwdriver supplied with the mounting kit to the three holes on the back of the speaker.

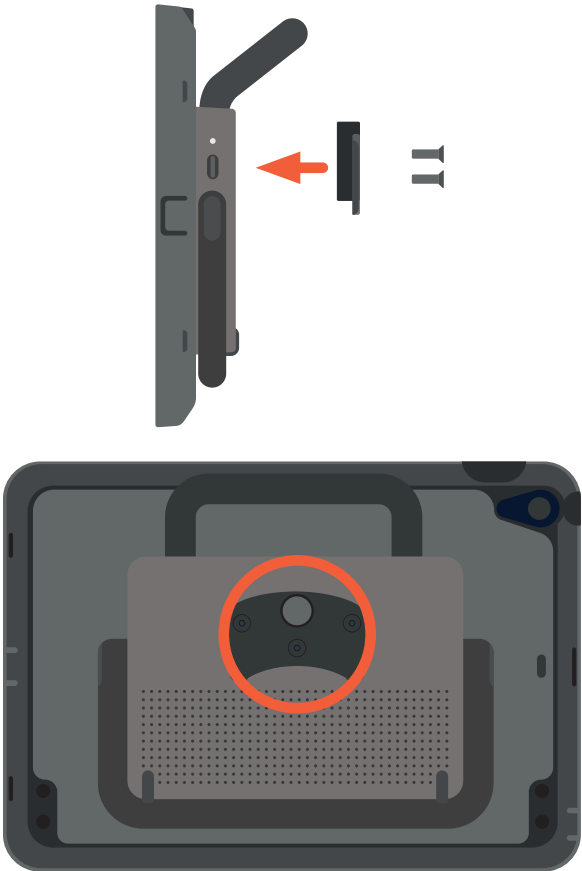

7. Attaching the strap

To attach the strap to your L 10, loop the end of the strap through the attachment point, unclip the connector and push it through the loop. Clip the connector again and repeat on the opposite side.

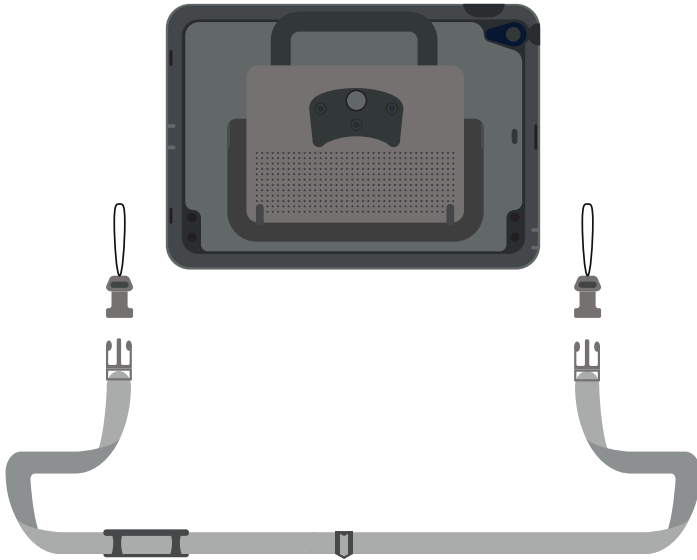

8. Finding you model and serial number

When you contact our Support team, please have your device model and serial number to hand. This helps us identify the device you are having an issue with!

You can find both the model number and serial number on the back of your L 10.

The model and serial number for the device is placed under the stand, lift the stand and the serial number is found here:



9. Firmware updates

From time-to-time Smartbox will issue firmware updates for the L 10 speaker.

To update the firmware, download the Smartbox Link app from the App Store.



hub.thinksmartbox.com/smartbox-link

Open the app, if a firmware update is available you will see a button to download and update the speaker.



L 10 (10.2) - ST - Manual - US - A