

L 8

Slim Edition

Manual



Communication is life

This device is manufactured by Smartbox Assistive Technology Ltd.

**United States**

Smartbox Assistive Technology Inc  
167 Hillcrest Shopping Center  
Lower Burrell, PA 15068

Document: L 8 - SL - Manual - US - A  
Device: TS08E-L8  
Language: English  
Date of issue: September 2023


Copyright © 2023 Smartbox Assistive Technology

# L - Series

## L 8 Slim Edition

This guide provides the basics for getting started with your L 8.

You can find a complete guide to all the features of your L 8 on the Smartbox Hub:

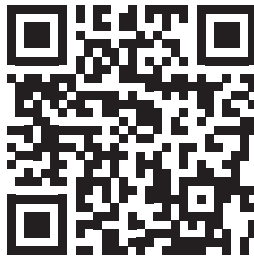


[hub.thinksmartbox.com/l-series](https://hub.thinksmartbox.com/l-series)

# 1. Setting up your device

## Step 1

Power on your L 8 with the power button on the left side of the device.

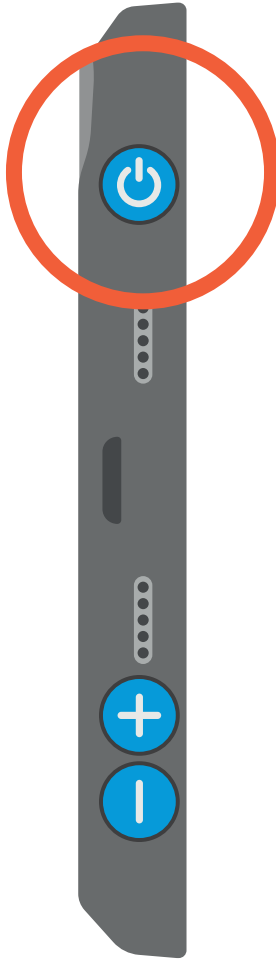

## **Step 2**

L 8 can be provided Ready to Go or for Self Setup


A short guide is provided to help you with your configuration

Remember to take note of your account details!

## 2. Power

The device power button is found on the top left side of your L 8.

To turn on your device on, press and hold the power button for 3 seconds.

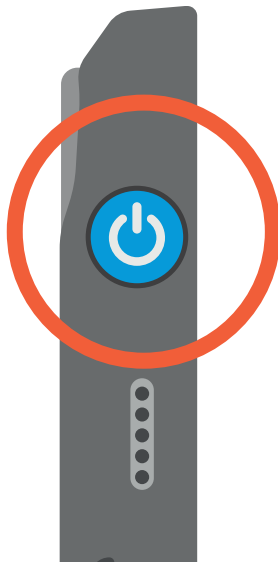

### **Putting your L 8 to sleep**

Sleep is a low powered mode where the device is inactive, and the screen is off.

Tap the power button to sleep the device.

Tap the power button again to wake the device.

## **Power off**

To turn the device off, press and hold the power button and the volume up button.

Swipe the 'Slide to power off' toggle on the screen. Press the cross to cancel.

### 3. Charging

Connect the L 8 power adapter to a mains socket.

Connect the charging cable to the power adapter.

Connect the USB C cable to charging port 1.






# 4. Mounting


Your L 8 can be mounted to a Rehadapt or Daessy connection (Mounting plate sold separately).

The dual mounting plate can be attached with the screws and screwdriver supplied with the mounting kit to the three holes on the back of the device.

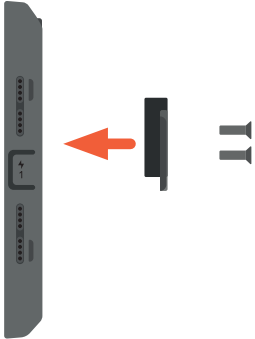

# 5. Attaching the strap

To attach the strap to your L 8, loop the end of the strap through the attachment point, unclip the connector and push it through the loop. Clip the connector again and repeat on the opposite side.

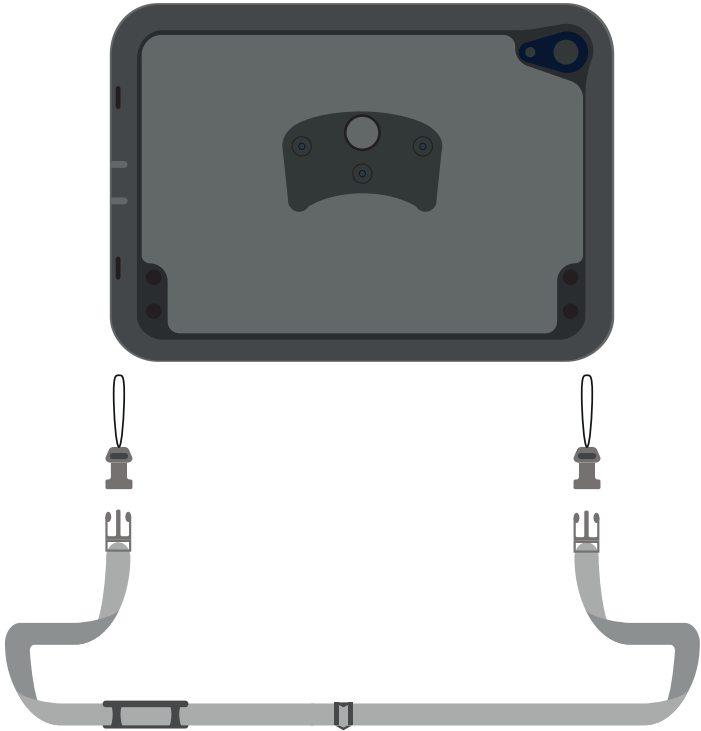

# 6. Finding you model and serial number

When you contact our Support team, please have your device model and serial number to hand. This helps us identify the device you are having an issue with!

You can find both the model number and serial number on the back of your L 8.





L 8 - SL - Manual - US - A