



L - Series

Keyboard quick start

Follow the steps below to fit your keyboard.

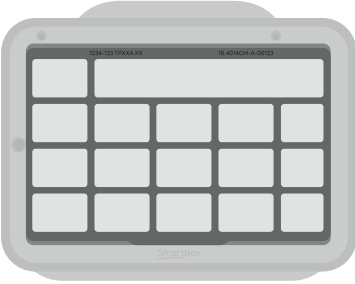

Place the perspex keyboard guard on the front of the device.

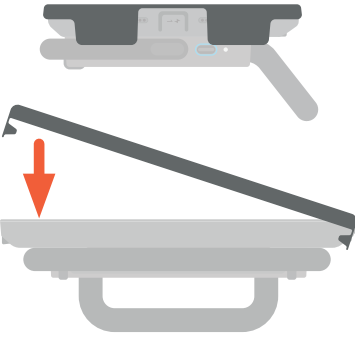








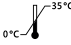
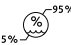


Place the right hand side of the frame with the two clips over the right hand side of your device and push the left side down to clip the device. You may need to pull the left clip outwards to fit correctly.



Push all four corners firmly to make sure the keyboard is secured.

Symbol explanation

Symbol	Meaning
	Conformity European symbol to declare conformity with EU legislation.
	Read the manual symbol
	UK Conformity Assessed symbol to declare conformity in the UK
	Unique device identifier
	Medical device
	EU Authorised Representative
	Swiss authorised representative
	Model number
	Serial number
	Min / Max storage and transport temperature
	Mix / Max storage and transport humidity

Cleaning and decontaminating your device

- Using a damp cloth or alcohol wipe, clean all the external surfaces.
- Allow the device to dry.
- Tough dirt and residue can be removed using a toothbrush or similar.

Do not use spray cleaning fluid, gel, or polish directly onto your device, or immerse the device in water.

This keyguard is manufactured by Smartbox Assistive Technology Inc.

US: Smartbox Assistive Technology Inc, 167 Hillcrest Shopping Center, Lower Burrell PA, 15068

Model numbers: 10090-140 / 10090-141 / 10090-142

Compatible with:

10090-140	10090-141	10090-142
TP08E-L8	TP10E-L10	TP10H-L10
TS08E-L8	TS10E-L10	TS10H-L10

Get support at hub.thinksmartbox.com