



Talk Pad

Keyguard quick start


Get the best start with your new keyguard accessory.

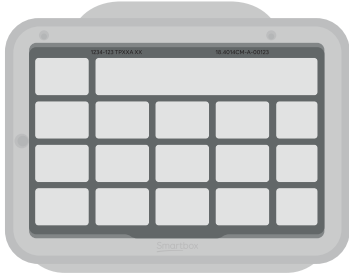

Place the perspex keyguard on the front of the device.

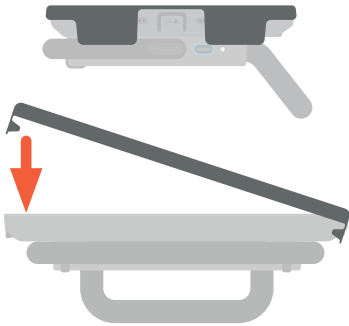



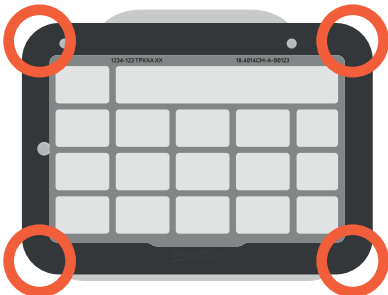


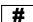

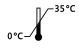
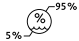


Place the right hand side of the frame with the two clips over the right hand side of your device and push the left side down to clip the device.



Push all four corners firmly to make sure the keyguard is secured.

Symbol explanation

Symbol	Meaning
	Conformity European symbol to declare conformity with EU legislation.
	Read the manual symbol
	UK Conformity Assessed symbol to declare conformity in the UK
	Unique device identifier
	Medical device
	EU Authorised Representative
	Swiss authorised representative
	Model number
	Serial number
	Min / Max storage and transport temperature
	Mix / Max storage and transport humidity

Cleaning and decontaminating your device

- Using a damp cloth or alcohol wipe, clean all the external surfaces.
- Allow the device to dry.
- Tough dirt and residue can be removed using a toothbrush or similar.

Do not use spray cleaning fluid, gel, or polish directly onto your device, or immerse the device in water.

This keyguard is manufactured by Smartbox Assistive Technology Ltd.

UK: Smartbox Assistive Technology Ltd Ysobel House, Enigma Commercial Centre Sandys Road Malvern WR14 1JJ.

EU: REHAVISTA GmbH, Konsul-Smidt-Str. 8c, 28217 Bremen, Germany

Model numbers: 10090 140 / 10090-141 / 10090-142

Compatible with:

10090-140	10090-141	10090-142
TP08E	TP10E	TP10H
TS08E	TS10E	TS10H
TP08E - CASE	TP10E - CASE	TP10H- CASE

Get support at hub.thinksmartbox.com